

## **BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING**

**INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT**

**APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### **INTENDED USE/INDICATIONS:**

The Aircast A60™ Ankle Support is designed to provide support and protection for ankles.

### **CONTRAINDICATIONS: N/A**

**WARNINGS AND PRECAUTIONS:** If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### **APPLICATION INFORMATION:**

1. Put on sock

- Unfasten strap from black plastic ring, open top flaps, and open brace


- Slide foot, toe first, into top of brace, Pull brace stabilizers upward and align stabilizers with sides of ankle

2. Wrap flaps around top of ankle and secure

3. Insert end of strap into black plastic ring

- Bend knee and align over toes

- Pull strap until snug and secure strap end onto body of strap

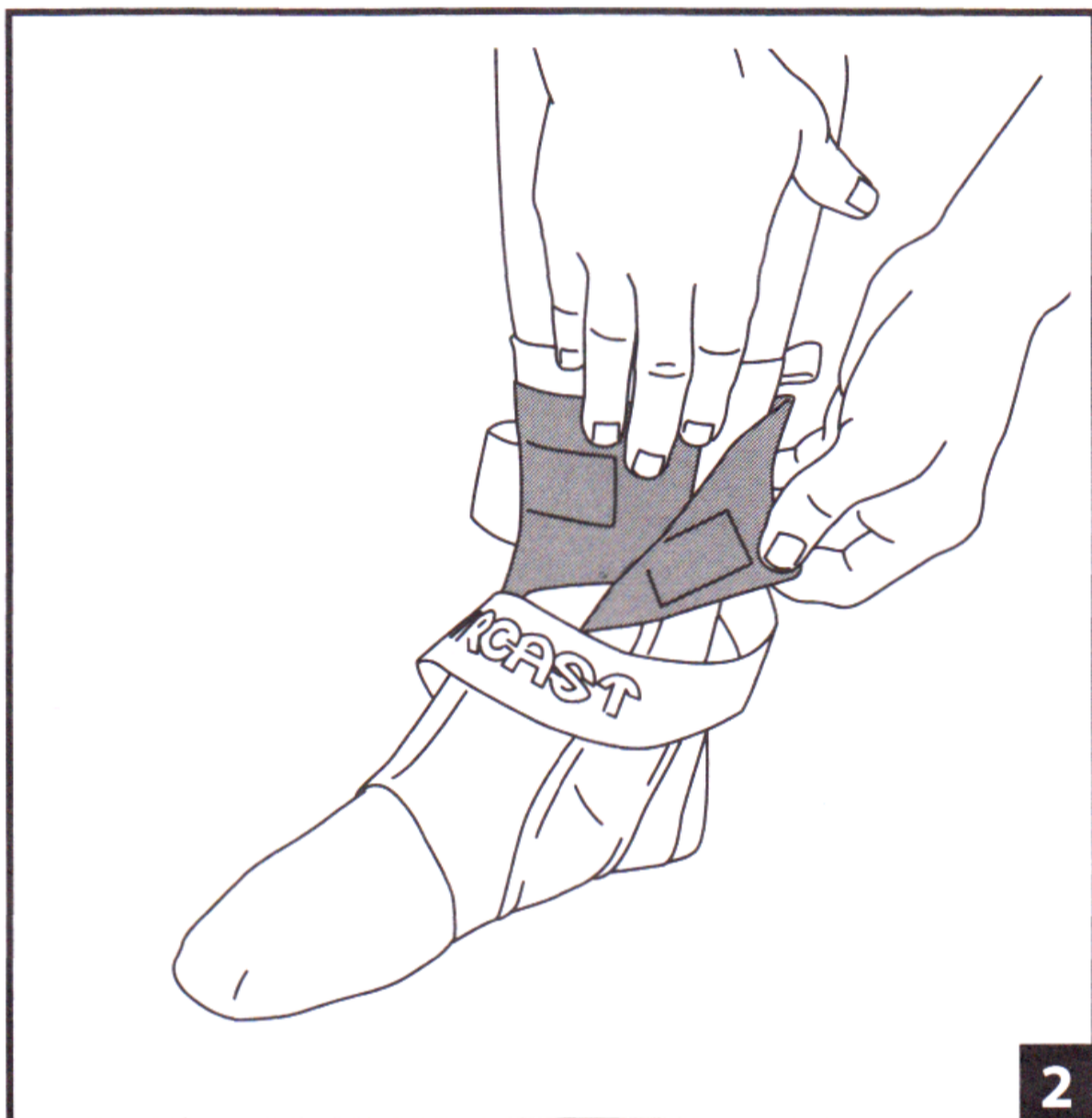

### **CLEANING INSTRUCTIONS:**

The entire A60 Ankle Support can be hand washed in lukewarm water with mild soap and air dried.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For single patient use only.

**LATEX FREE**

