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Description

This support provides post operative immobilisation or ROM control for the elbow. Providing a conservative treatment of elbow dislocation or luxation, this sturdy and adjustable support can also support stable or internally fixed fractures of the distal humerus, proximal radius or ulna. ROM can be limited by adjusting the simple hinges.

This support is manufactured from high quality materials and we recommend that a medical professional fits the support for you. If at any time you experience discomfort or your condition worsens, remove the support immediately and consult your therapist/medical professional.

Fitting Instructions

- Locate the hinge pivot centre over the lateral humeral epicondyle (outer elbow).
- To adjust length, press the button on the distal cuffs and pull the cuff away from the elbow joint to the desired length, to decrease pull towards elbow joint. The lower extension (ulna side of arm) should extend up to the styloid process (the bump on the medial side of the wrist). The upper extension (humeral side of arm) should extend up to the most comfortable position at the axilla or arm pit.
- Bend the cuffs to securely wrap around the circumference of the arm. Start with the cuffs proximal to the elbow.
- To adjust range of motion, press the flexion or extension button and rotate until the desired degree of flexion or extension shows through the corresponding window. Brace provides full range of motion from 0 to 120 degrees.

Cleaning and Care

Hand wash in warm water with liquid mild soap and leave to dry naturally.

**DO NOT MACHINE WASH OR TUMBLE DRY
NEVER OVER TIGHTEN STRAPS.**

This support should feel comfortable at all times.
REMOVE BEFORE GOING TO SLEEP.

Manufactured for: Patterson Medical Ltd, Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 100 www.pattersonmedical.co.uk

International: Tel: +44 1623 448 706 Fax: +44 1623 448 784

Patterson Medical Products, Inc. 28100 Torch Parkway Suite 700, Warrenton, IL 60555-3938, USA.

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